



Synergy^{ID}™ System

Quick Set Up Guide

Product Items

SYNERGY^{ID} SYSTEM

Synergy^{ID} Console, Tablet, Autoclavable Camera Head, and Cables



SYNERGY LASER LIGHT SOURCE



SYNERGY TABLET AND DOCKING STATION



VIDEO CART AND BOOM ARM (Optional)



SYNERGY MONITOR

Arthrex Synergy 4K Monitor and Cables



PRINTER (Optional)

Sony™ UP-DR80MD Printer and Cables



Synergy^{ID} Quick Set Up Guide

1. Place Synergy^{ID} console (CCU) on tower shelf and connect power cord to the back of the console
2. Attach monitor to the tower and connect monitor DC power cable to the rear panel of the monitor
3. Attach Synergy tablet to tablet arm and connect the cable from the docking station to the rear panel of the Synergy console labeled "TABLET" as shown
4. Using the provided video cable, connect one end to the rear panel of the console labeled "DISPLAY PORT 1" as shown
5. Connect the other end of the video cable to the rear panel of the monitor labeled "VIDEO INPUT"
6. Attach provided trigger cable from back of laser light source to the console labeled "LASER" (only needed if fluorescence imaging is desired)
7. Connect the laser light source to power (only needed if fluorescence imaging is desired)
8. Place printer on tower shelf and connect power cord to power supply and USB cable to the rear of the Synergy^{ID} console
9. Press the LED "ON" button on the front of the CCU to turn the light source on

