## Cleaning and Sterilization Directions for Use for Reusable Connection Devices for Use with AR-1650 Shoulder Suspension System

AR-1650-02 Arm Sleeve Connector

AR-1650-03 Lateral Traction Sleeve Connector

The Cleaning and Sterilization directions apply to the S3 Arm Sleeve Connector (AR-1650-02) and the S3 Lateral Traction Sling Connector (AR-1650-03). For further directions for use regarding the Arthrex® Shoulder Suspension System please refer to the S3 System IFU (D-720764).

**WARNING:** Failure to properly sterilize device could cause exposure to contaminant.

### Automatic Cleaning Procedure

#### AR-1650-02 & AR-1650-03

1. Prepare the detergent Neodisher® Mediclean Forte at ¼ oz/per gallon of lukewarm tap water in an ultrasonic bath.
2. Fully immerse the devices in the prepared detergent and sonicate for 10 minutes.
3. Remove the devices from the detergent and transfer them into the washer for processing.
4. The washer parameters that have been validated are shown below in Table 1. The motor speed should be set to high.

Manufactured For:

Arthrex, Inc.  
1370 Creekside Blvd  
Naples, FL 34108, USA  
1-(800) 934-4404

Arthrex GmbH  
Erwin-Hielscher-Strasse 9  
81249 München, Germany  
+49 89 909005-0

Allen Medical Systems  
100 Discovery Way  
Acton, MA 01720  
(978) 266-5401

Hill Rom SAS  
B.P. 14 Z.I. du Talhouet  
56330 Pluvigner, France  
Tel. +33 (0) 2 97 50 92 12  
Fax. +33 (0) 2 97 50 92 00

D-720771-A1  
08 FEB 2016

**Table 1: Validated Washer Parameters for the AR-1650-02 & AR-1650-03.**

Phase	Recirculation time (minutes)	Temperature	Detergent type and Concentration
Pre-Wash 1	3:00	Cold tap water	N/A
Wash 1	10:00	65.5°C	Neodisher® Mediclean Forte ¼ oz/gallon
Rinse 1	3:00	50°C	N/A
Rinse 2	3:00	50°C	N/A
Thermal Disinfection	5:00	90°C	N/A
Drying	6:00	115°C	N/A

**Steam Sterilization for AR-1650-02 & AR-1650-03**

**Sterilizer Type:** Prevacuum  
 Preconditioning Pulses: 4  
 Minimum Temperature: 132°C  
 Full Cycle Time: 4 minutes  
 Minimum Dry Time: 30 minutes  
 Cool-Down Time: 30 Minutes outside of chamber on a wire rack

**Sterilizer Type:** Gravity  
 Minimum Temperature: 132°C  
 Full Cycle Time: 15 Minutes  
 Minimum Dry Time: 50 Minutes  
 Cool-Down Time: 30 minutes outside of chamber on a wire rack

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AR-1650-02 *Arm Sleeve Connector*

AR-1650-03 *Lateral Traction Sleeve Connector*

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